



D Das **BORT Silikon-Fersensporenpolster** mit SoftSpot bietet eine spürbare Druckentlastung beim sogenannten Fersensporn, einem dornartigen Knochenauswuchs an der Unterseite des Fersenbeines.

Anwendung: Mit der seitlichen Randerhöhung nach oben in den Fersenbereich des Schuhs einlegen. Dabei auf rechte (R) und linke (L) Ausführung achten. Das weiche Material passt sich der Kontur im Fersenbereich optimal an und haftet sicher, ohne zu verrutschen.

Pflegehinweis: Zur Reinigung mit Seife waschen und gut trocknen lassen. Oberseite leicht einpudern.

GB The **BORT Heel Spur Cushion** with SoftSpot provides a considerable pressure relief in case of so-called heel spur, a thorn shaped exostosis at the lower side of the heel bone.

Application: Put into the heel area of the shoe with raised periphery turned upward. Please mind the right (R) and the left (L) version. The soft material adjusts itself optimally to the contour in heel area and stays safely without slipping.

Care Instructions: For cleaning wash with soap and dry well. Powder top slightly.