PECTOCOR® M

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RIEMSER

PECTOCOR® M

Medical device to support massage in the segmental therapy in the mammary region

INSTRUCTIONS FOR USE

1. Therapeutic purpose

Pectocar® M is a massage cream applied to support the physical segmental therapy for the uninjured skin in the mammary region.

2. Therapeutic indications

As treatment method of physical medicine segmental therapy has a positive influence on interior organs such as the heart from the application of mechanical cutaneous stimulants (by means of massage, heat, cold, acupuncture). The segmental therapy assumes that certain cutaneous areas (so-called Head's zones) are connected to internal organs by nerve tracts. Disease processes can thus be influenced by the vegetative nerve tracts via local or super-ordinate closed loops.

Pectocar® M is suitable both for the application at home by adults or for the professional treatment of patients, the handicapped and the elderly.

3. Undesirable effects

After the application of *Pectocar® M* local irritations or contact eczema may occur. A temporary redness of the skin in the range of application is, however, due to the preparation. Bronchial spasms may be increased. If hypersensitive reactions to the ingredients of the cream occur, stop using *Pectocar® M*.

4. Histocompatibility, Interaction

Pectocar® M should not be applied at the same time as other creams and ointments to avoid a possible interaction.

5. Contraindications

Pectocar® M may not be applied if the patient suffers from spasmodic asthma and other respiratory tract diseases which are accompanied by a distinct sensitivity of the respiratory tracts. The product may not be applied either in the presence of skin diseases, to injured skin or sores.

6. General special precautions

Pectocar® M is solely designed for external application on uninjured skin and may not come into contact with eyes, mucous membranes or sores.

Attention:

Wash hands thoroughly with water and soap after the treatment.

Attention:

Pectocar® M may exclusively be applied by adults. The application to nurslings and children under the age of 16 and to pregnant and breast feeding women is excluded.

Attention:

Keep out of the reach and sight of children!

Attention:

The product is not to be applied when hypersensitivity to one of the ingredients is known.

Attention:

Before taking *Pectocar® M* out of its packaging, check the latter for integrity.

7. Product properties

Pectocar® M contains various natural herbal extracts such as fir leaf oil, eucalyptus oil, menthol, mint oil and camphor, which, when spread gently

to the range of application, produce a pleasant and refreshing feeling in the thoracic and cardiac region.

On the other hand, a strong massage of the application range with the product and the volatile odorous substances and excipients, produces a clearly noticeable, pleasant sensation of heat on the skin.

8. Product application

Pectocar® M is rubbed in gently to the skin of the mammary region over the cardiac region to produce a refreshing feeling or is massaged strongly to produce a pleasant sensation of heat. Lengths of 1-2 cm are squeezed out of the tube 1-3 times a day.

Pectocar® M is designed for long-term application, i.e. until the symptoms subside.

After squeezing the product from the tube, close the latter tightly again and wash hands thoroughly after treatment!

The used tube and folding box are disposed of via domestic waste. Preferably separated.

9. Physical product data

Composition:

White petrolatum, purified water, lanolin, hard paraffin, hard paraffin, sorbitan sesquioleate, cetyl palmitate, spruce needle oil, cetostearyl alcohol, eucalyptus oil, menthol, mint oil phenoxyethanol, 1.5 % racemic camphor as odorous substance.

Appearance, smell: Yellowish homogenous ointment with slight, characteristic smell of camphor.

Self-life: 2 years

The minimum shelf-life is printed on the welded fold at the bottom of the tube, on the folding box of the product respectively.

Storage conditions:

The product is stored at room temperature not exceeding 25 °C. Close tightly after use.

10. Delivery specifications

25 g tube, packed in folding box

50 g tube, packed in folding box

11. Manufacturer / Distributor

Manufacturer and distributor of the medical device:

 RIEMSER Arzneimittel AG
An der Wiek 7
17493 Greifswald-Insel Riems
fon + 49 (0) 38351 76-0
fax + 49 (0) 38351 76-48

 Manufacturer  Temperature limits  Shelf life

 Batch definition/Batch number  Note Instructions for Use

Note:

Please read and comply with instructions for use.

This leaflet was last approved in 10/2009.

MEDIrecycling 

1141376/12
Code 324

RIEMSER 