**Größen / Sizes /
Dimensions / Tamaños**

	Umfang Hals/Circumference of neck/ tour du cou/Extensión alrededor de cuello	Kinnhöhen/Height of chin/ Hauteur du menton/Altura de la barbilla
small	bis/up to/jusqu'à/hasta 38 cm	7,5 cm
medium	38 - 43 cm	9,0 cm
large	über/over/plus de/sobre 43 cm	11,0 cm

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ⒼⒷ

Injuries in the cervical vertebra area are effecting a complex structure of ligaments, muscles, sinews, nerves and vessels. Immobilization support recovery of these tissue structures and helps at the same time avoiding to a high degree mismovements.

Instruction for use:

Open up cervical support and grip it with both hands horizontally. The velcro closure strap is placed at the back.

Put on cervical supports in a way that the recess fits directly below the chin. Then wrap the support around the neck and lock the velcro closure at the rear side. Place the cervical support in a way that it supports but does not strangle. The support is made of soft foam (density 45) and therefore can also be worn at night.

Indications:

For whiplash injury of the cervical vertebra column aching tilted neck, after operations of the cervical vertebra column and after operations of the tilted neck. Cervical vertebra column syndroms.

Important Instructions:

No reutilization – this medical device is meant for the care of one patient only.

In case of doctor's order to wear bandage during night, avoid hindering blood circulation. In case of numbness loosen bandage and remove it if necessary. If complaints continue, please contact the doctor.

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